iCare361 Personal Blood Pressure Log Function Operational Guidelines

kutech KUO YUE TECHNOLOGY CO.,LTD.

www.kutech.com.tw

The software can have inexhaustible innovation and creation; therefore, it's enchanting. The recent technology develops in order to meet the user's needs. The renovation of the consumer software is one good example.

The software iCare361 broke new ground in the realm of blood pressure managing system. KUO YUE would like to introduce the personal blood pressure reading solutions in iPhone. To record your blood pressure reading from your home blood pressure monitor. Our system allows you to enter your systolic, diastolic and pulse rate reading by date and time. In addition, you can also record any notes about how you are feeling or other important information. KUO YUE always persists in giving consideration to both technology innovation and quality.

iTunes Link:

• iCare361

Features:

- System develop based on iOS5
- Fast & easy to setup iCare361 software
- Great user-interface
- To record your systolic, diastolic, pulse rate, important information
- Data transmission with e-mail
- **Export a text file** of your blood pressure data if you need a delimited text file to load into another program.
- Export your blood pressure information directly to Excel (CSV) if you're not sure what info you need and you'd rather delete and rearrange columns in a spreadsheet program.
- To display date range on charts header
- The charts shows the accumulated total of counts
- Computing and display bar charts statistics

1.Main Menu

iPad ᅙ		上午11:18	100% 🖙
Cancel		Blood Pressure Log	Apply
Systolic mmHg	Diastolic mmHg	Pulse	
Information			
Date	Sdp	Bdp Pulse	
2012-04-07,	11:17 142	076 072	
	Input Date	Setup Statistics Export	About

Input Date

iPad 奈			上午11:18				100% 🛃
Cancel		Blood	Pressure L	og	_	_	Apply
Systolic	Diastolic	Pulse		4月5日 週四	9	9	16
mmHg	mmHg		4	4月6日 週王	ī	10	17
Information				Today	上午	11	18
			4	4月8日 週日	下午	12	19
				4月9日 週一		1	20
Date	Sdp	Bdp P	ulse				
2012-04-07,	11:17 142	076 0	72				
					_		
	Input Date	Setu	p Statistics	Export Abo	out		_

2.Export Function

The following procedure description how to export information from ICARE360. While other export options are available, and briefly discussed, the instructions focus on the **Export as Text** option

Under the Toolbar menu, selected the desired **Export** option.

Export as Text File is the recommended option. It will export the selection to a text (.txt) file, which can be opened in a variety of application.

CSV button

Export as CSV File is the recommended option. It will export the selection to a csv (.csv) file. **Clear** button

Clear data: All of this stock count data will be deleted permanently.

Blood Pressure Log	
Pl	ease choose the type of data that you.
	Export as Text file
	Export as CSV file
	Clear data

3.Setup

d Pressure Log	Setup	
	octup	
First name	Steven	>
Last name	Lin	
Age	40	
Gender	male	
Send	service@kutech.com.tw	>
То	sales@kutech.com.tw	
Cc	service@kutech.com.tw	
Bcc		
Subject	Personal Blood Pressure Log iCare361	
Content	Wish you good health!	>
Guidelines		>
Factory	Default	

3.Setup/Guidelines

iPad ᅙ	上午11:39	100% 🖙
Setup	Guidelines	
[Statistics]		

[Statistics] Start:Statistical data.

[Export]

Export as Text:It will export the data to a text(.txt) file;by default, the file is named 123.txt. Export as CSV:It will export data to a excel(.csv) file;by default, the file is named 123.csv. Clear data:All of this blood pressure data will be deleted permanently.

[Setup] First name:Your first name. Last name:Your last name. Age:Your age. Gender:Female or Male Send: To: Cc: Bcc: Subject: Content: Guidelines:Operational guidelines. Factory:It's default factory settings.

4.Statistics

۲

Tab **Statistics** button to Computing and display the Bar Chart statistics.

iPad 🔶 Blood Pres	ssure Log	上午11:20	100%
	Date:2012-04-07		
	Hypotension	< 90	1
	Desirable	90-119	1
	Prehypertension	120-139	1
	S1 Hypertension	140-159	2
	S2 Hypertension	160-179	1
	Hypertensive	>=180	1

Total: 7

4

5.About

iPad ᅙ	上午11:39	100% 🖼
Blood Pressure Log	關於	

kutech

iCare361 個人血壓日誌 Version:1.0.1.5 http://www.kutech.com.tw 版權所有 @2012 國祐科技股份有限公司

L.